
**USER MANUAL
MANUAL DE USUARIO
BENUTZERHANDBUCH
MANUEL D'UTILISATION
MANUALE UTENTE**

**KMINA - BATH TRANSFER BENCH
KMINA - BANCO TRANSFERENCIA
KMINA - BADEWANNEN TRANSFERBANK
KMINA - BANC DE TRANSFERT
KMINA - SEDIA PER IL TRASFERIMENTO**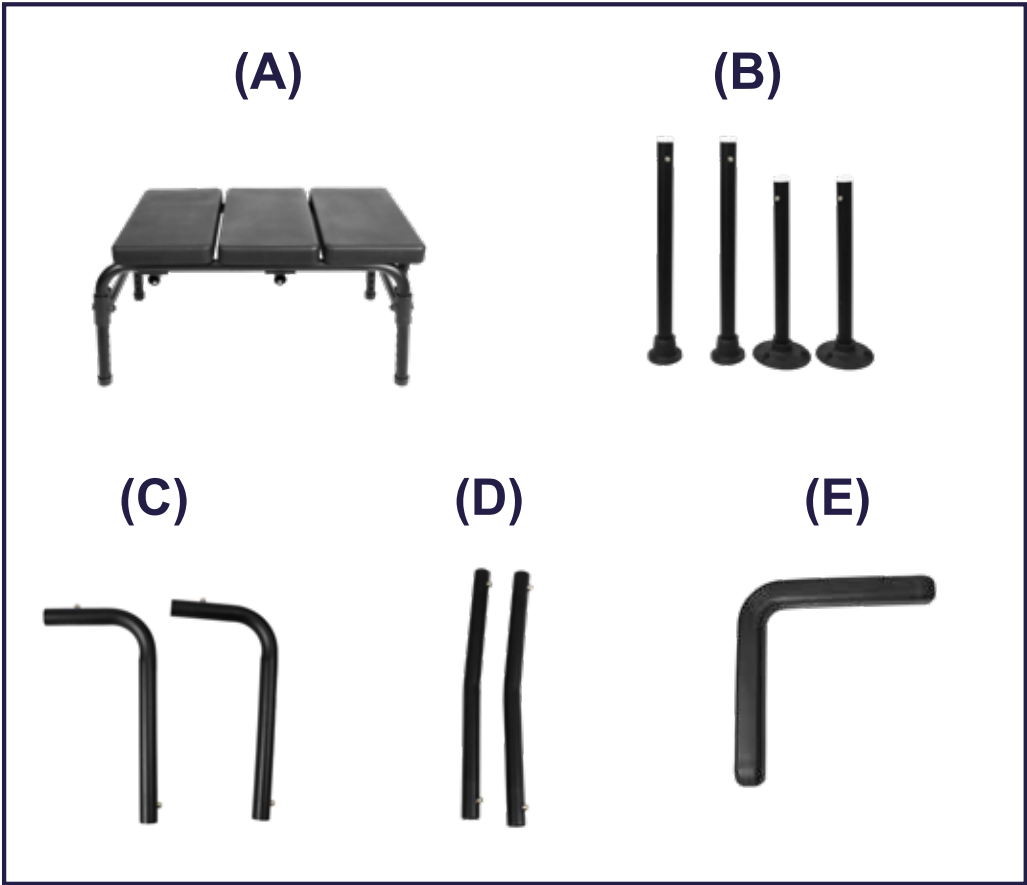

	ENGLISH	ESPAÑOL	DEUTSCH	FRANÇAIS	ITALIANO
A.	Seat	Asiento	Sitz	Siège	Sedile
B.	Legs	Patas	Beine	Jambes	Gambe
C.	Rear tubes	Tubos traseros	Hintere Rohre	Tubes arrière	Tubi posteriori
D.	Side tubes	Tubos laterales	Seitliche Rohre	Tubes latéraux	Tubi laterali
E.	Armrest	Reposabrazos	Armlehnen	Accoudoirs	Braccioli

1

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3

EN Insert the side tubes (D) into the holes on the side of the seat. Make sure you hear the click of the adjustment button indicating that it is correctly positioned.

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4

EN Finally, fit the armrest (E) by inserting the tubes into the holes. Make sure you hear the click of the adjustment button indicating that it is correctly positioned.

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The **TRANSFER BENCH** for bathtub is designed to facilitate access to the bathtub for the elderly, people with reduced mobility or people who have knee and/or hip problems. It should be used as an aid as long as there is sufficient torso stability. **BEFORE** using this product read the manual and keep it.

FEATURES

- Assembly without the use of tools.
- Ergonomic surface and drainage holes.
- Safe and stable thanks to non-slip suction cup feet.
- L-shaped armrests for easy turning of the user.
- Corrosion-free thanks to its aluminum chassis.

INSTALLATION

- You must adjust the legs and make sure they are locked and the seat is completely horizontal.
- **IMPORTANT:** Place the transfer bench with the handle and suction cups inside the bathtub for a secure grip.
- You can adjust the direction of the armrest depending on the orientation of the shower head.
- The narrow end caps will be located on the outside.

WARNINGS!

- Check all parts to verify if all components are present or if there was any damage during shipment.
- Make sure that each pair of legs are adjusted to the same height, with the bench remaining horizontal. Note that the inside of the bathtub or shower may differ in height with the floor outside.
- Make sure that the adjustments fit into the holes in the legs with a click.
- Do not use for transfers from the side or for partial loading.
- Do not sit on it unless all four legs are in full contact with the floor.
- **DO NOT** stand on the bench.
- Do not exceed the maximum supported weight which is 210 lbs.
- The user must be able to maintain balance when using without an assistant.
- The user must be careful not to insert his extremities (fingers, hands, genitals, feet and head) in any space between the different parts of the product.
- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.
- Do not remove the sticker provided on the product. If necessary, note the batch of the product so that the manufacturer can be notified in the event of an accident.

CLEANING AND DISINFECTION

The product can be cleaned and disinfected with commercially available products. Always keep the product clean and wash it regularly.

1. Remove the feet.
2. Use water and a mild, non-abrasive detergent to clean the separate parts.
3. Disinfect by wiping all accessible surfaces with disinfectant.

WARRANTY INFORMATION

KMINA warrants this device to be free from defects. The warranty will remain in effect for 3 years from the date of original consumer purchase. The warranty does not extend to parts exposed to natural wear and tear requiring periodic replacement. This warranty does not cover:

- Replacement parts supplied by anyone other than KMINA or the authorized dealer.
- Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
- KMINA is not liable for accidental damage or damage to property.

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