## Español

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## English

### INSTRUCTIONS FOR USE AND PRESERVATION

Dear customer,

Thank you for your trust in this Orliman product. You have acquired a quality product with a high medicinal value. Please, read the instructions carefully. If you have any doubt, get in contact with your doctor, your orthopaedist specialist or our customer service department. Orliman appreciates your choice and wishes you a speedy recovery.

#### REGULATION

These products satisfy the European Medical Device Directive 93/42/EEC (RD. 1591/2009). A Risk Analysis has been carried out (UNE EN ISO 14971) minimising all the existing risks. The trials have been carried out in accordance with the European regulation UNE-EN ISO 22523 of Prosthesis and Orthosis.

#### SUITABLE FOR

Protection and control following traumatism or surgical intervention, sprains and ligament injuries, as a method to replace plaster in the treatment of preulcerous or ulcerous conditions of the foot's plantar surface. Acute plantar fasciitis and in those cases which require a uniform distribution of body weight on the plantar surface and immobilisation of the tibia-tarsus articulation.

#### FITTING INSTRUCTIONS

**THE PRODUCT SHOULD BE ADJUSTED BY YOUR ORTHOPAEDIC TECHNICIAN OR IN ACCORDANCE WITH DOCTOR'S ORDERS.**

In order to obtain a greater degree of therapeutic effectiveness in the different pathologies and to prolong the life and utility of the product, it is fundamental that the correct size adequate for each patient or user is chosen (enclosed in the packet is a diagram with all the sizes and the equivalent in centimetres). An excess compression can produce intolerance, therefore we recommend that the compression be regulated to an optimum degree.

#### To fit observe the following aspects:

A+B- Position an interface, preferably of cotton, on the patient's leg and foot (sock).

Then position the surrounding cover, clasping the patient's foot and leg and making it as close fitting as possible using the Velcro seal system.

C- Next introduce the leg in the orthosis. Ensure the foot makes direct contact with the base, and that the mediolateral strips are parallel to each other and aligned on the leg and ankle load line. Attach them to the cover using the Velcro straps which are found inside the strips.

D- Finally position the tibial and calf protection plates, to then seal and adjust the orthosis using the Velcro straps.

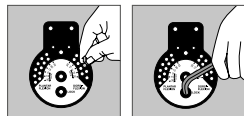
E\*- For model EST-083 only: Next, inflate the airbag to the desired pressure using the inflating pump.

To inflate, turn the valve towards INFLATE (clockwise). The valve must be tightly closed so that air does not leak. To let the air out of the bag, turn the valve towards DEFLATE (counterclockwise) until comfortable. Let air out if the lining is too tight. Remember to close the valve tightly towards INFLATE to avoid air leakage.

#### Instructions for the adjustment of the Walker articulation:

Slide and open the cover. Turn the cover of the articulation to make the pins accessible. Release the pins, select the required position and reinsert the pins. Close the cover. In order to lock the bars at an angle of 90°, position both pins at 0°. In order to lock at 7.5°, 15°, 22.5° or 30° of plantar or dorsal flexion, position the flexion pins of each articulation in the required position, and then proceed in the same way with the extension pins of each articulation. When you wish to lock a fixed position at 0°, 7.5°, 15°, 22.5° or 30° of plantar or dorsal flexion, tighten the Allen screws of both articulations.

Note: In order to proceed with the regulation of the extension flexion, first loosen the locking screw.

#### ATTENTION:

To ensure the correct functioning of the orthosis, after inflating, the valve should be closed tightly so that it does not allow air to escape. To inflate, turn the valve clockwise.

#### PRECAUTIONS

The constructive material is inflammable.

Do not expose these products to situations where they could ignite. If this were the case remove them quickly and use the necessary measures to put it out. The material used is hypoallergenic, however we cannot guarantee 100% that in some determined cases it will not produce allergies, if this occurs, remove the product and see the doctor that prescribed it.

In case of minor irritations due to sweat, we recommend the use of a cotton sock, to separate the skin from contact with the material. For small inconvenience caused by sweat, we recommend using an interface to separate the skin from contact with the tissue. If rash, irritation or swelling to remove the product and consult a doctor or prosthetist. Contraindicated in open scars with swelling, redness and accumulation heat.

### RECOMMENDATION-WARNINGS

These products are to be used only by those which have been medically prescribed to do so. Persons unconnected to such prescriptions should not use them. When the product is not being used, store in its original packaging. In order to dispose of the packaging and the product, strictly comply with the legal norms of your community.

The use of the orthosis is conditioned to the recommendations of the prescribing doctor, therefore it must not be used for any other means than those prescribed.

For the validity of the guarantee, the issuing establishment should comply with this sheet of instructions.

#### MANUFACTURE-CHARACTERISTICS

The material features used in production have been tested and approved and meet all the European quality requirements.

All the products are produced with high quality materials and offer an unbeatable comfort and quality of use. All products offer restraint, stability and compression for the optimum treatment of the pathologies for which they have been designed.

#### RECOMMENDATIONS FOR PRESERVATION AND CLEANING

Stick the velcros to each other (if the orthosis has any), wash periodically by hand with a damp cloth and a neutral soap. To dry the orthosis, use a dry towel to absorb most of the dampness and leave to dry at room temperature. Do not hang up or iron and do not expose to direct heat sources such as stoves, heaters, radiators, direct sun light etc. During use or during washing do not use alcohols, ointments or dissolvent liquids. If the orthosis is not properly dried any detergent residues could irritate the skin and deteriorate the product.

#### GUARANTEES

ORLIMAN, S.L.U. Guarantees all its products, as long as these have not been manipulated or altered from their original state. Does not guarantee those products in which as a result of misuse, deficiencies or breakage of any kind, its original state has been altered. If you observe any deficiency or anomaly, communicate this immediately to the establishment from which it was obtained in order for it to be changed.

ORLIMAN, S.L.U. Appreciates your choice and wishes you a speedy recovery.

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