EN	REFERENCE	
	PEPE - WHEELCHAIR RAMP 12"	P10040
	PEPE - WHEELCHAIR RAMP 23"	P10041

The **PEPE WHEELCHAIR RAMP** has been designed designed to overcome obstacles, especially steps.

INSTRUCTIONS FOR USE

- Position the ramp so that it passes over the step where you wish to place it.
- The ramp must rest firmly on the level surface.
- Anchor the ramp to the pavement through the two marked holes.
- Screws are included for anchoring to a wooden or plastic surface (A). If the surface is concrete, first drill the holes in the pavement and then use plugs (C) and suitable screws (B).
- Ensure that the ramp does not move during use.

WARNINGS

- Remove and retain all packaging.
- Make sure that the ramp has not been damaged during transport or storage.
- Read the instructions and warning texts carefully before using the ramp.
- Keep the instructions for reference.
- Ensure that the surface at both levels is firm, stable and capable of supporting the intended load.
- The ramp is not designed for permanent outdoor use.
- Any modifications/repairs that have not been carried out by the manufacturer will void the warranty.
- Check that the ramp is firmly positioned and stable each time it is used.
- Take care when using the ramp, no part of the wheelchair should catch and move the ramp.

CLEANING AND MAINTENANCE

- If the ramp is damaged or becomes slippery in any way, it should not be used (contact supplier/manufacturer).
- Check the anchoring to the pavement periodically.
- It should be cleaned with a damp cloth with soap.
- Do not use corrosive cleaning agents.

CHARACTERISTICS

- Non-slip surface.
- Includes two screws (no dowels included).
- Reflective strips on both sides.

WARRANTY

IMPROVING MOBILITY DEVELOPMENT SL. guarantees that this device is free from defects. The warranty will remain in effect for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement.

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